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### OKKLUGlas und OKKLUGlas AERO

#### Anwendung:

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### OKKLUGlas and OKKLUGlas AERO

#### Use:

 The watch glass bandage (74 mm x 93 mm x 18 mm) is intended for single use only.

 Store dry

**To ensure optimal adhesion, the skin should be clean, free of grease and dry.** Bushy eyebrows can affect the adhesion of the patch: Shave or pluck the eyebrows if necessary. Otherwise also available in the EXTRA variant: super hold due to extra large comfort patch (90 mm x 93 mm x 18 mm). Please remove the cover paper carefully. Place the watch glass bandage centrally on the eye so that the break-proof protective cover is centered over your eye and your eye is not covered by the plaster. Fix the patch to the skin with gentle pressure. Please consult your doctor about the medication and the optimal variant.

#### For OKKLUGlas

##### Intended purpose:

Ready-to-use eye bandage with transparent, break-proof protective cover: hypoallergenic, ergonomic and extremely skin-friendly

##### Indication:

To be applied after ptosis surgery, for lagophthalmus, facial nerve paresis, to treat injuries to the outer eye, for protection after eye surgery.

##### Contraindications:

Allergy to plasters / plaster intolerance.

##### Effect:

OKKLUGlas creates a „moisture chamber“.

##### Warnings:

**Spatial vision will be severely restricted when wearing a watch glass bandage (roadworthiness). The plaster must not be adhered to injured skin or mucous membranes.**

#### For OKKLUGlas AERO

provides optimal air circulation through ventilation openings

##### Intended purpose:

Ready-to-use eye bandage with transparent, break-proof protective cover: hypoallergenic, ergonomic and extremely skin-friendly

##### Indication:

To be applied as mechanical protection of the eye after ptosis operations, for lagophthalmos, and after eye surgeries.

#### Contraindications:

Allergy to plasters / plaster intolerance.

#### Effect:

Mechanical barrier feature; OKKLUGlas AERO provides air circulation through ventilation openings.

#### Warnings:

**Spatial vision will be severely restricted when wearing a watch glass bandage (roadworthiness). The plaster must not be adhered to injured skin or mucous membranes.**

Serious incidents occurring in connection with the product must be reported to us as the manufacturer and/or the competent authority of your member state.

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Berenbrinker Service GmbH