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Size: kids: 70 x 58 x 14 mm | baby: 48 x 39 x 11 mm

Use:

The watch glass bandage is intended for single use.

Keep dry during storage and protect from sunlight.

To ensure optimal adhesion, the skin should be clean, free of grease and dry. Bushy eyebrows can affect the adhesion of the patch: Shave or pluck the eyebrows if necessary. Carefully remove the blue cover liner. Place the watch glass bandage centrally on the eye so that the break-proof protective cover is centered over your eye and your eye is not covered by the plaster. Fix the patch to the skin with gentle pressure. Please consult your doctor about the medication and the optimal variant.

For OKKLUGlas® kids SILICONE & OKKLUGlas® baby SILICONE

Intended purpose:

Ready-to-use eye bandage with transparent, break-proof protective cover: hypoallergenic, ergonomic and extremely skin-friendly

Indication:

To be applied after ptosis surgery, for lagophthalmus, facial nerve paresis, to treat injuries to the outer eye, for protection after eye surgery.

Contraindications:

Allergy to plasters / plaster intolerance.

Effect:

OKKLUGlas® kids SILICONE & OKKLUGlas® baby SILICONE creates a „moisture chamber“.

Warnings:

Spatial vision will be severely restricted when wearing a watch glass bandage (roadworthiness). The plaster must not be adhered to injured skin or mucous membranes.

Patient target group: Babies and children with the support of an adult. Please follow your doctor's instructions and select the variant and appropriate size.

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provides optimal air circulation through ventilation openings

Intended purpose:

Ready-to-use eye bandage with transparent, break-proof protective cover: hypoallergenic, ergonomic and extremely skin-friendly

Indication:

To be applied as mechanical protection of the eye after ptosis operations, for lagophthalmos, and after eye surgeries.

Contraindications:

Allergy to plasters / plaster intolerance.

Effect:

Mechanical barrier feature; OKKLUGlas® kids AERO SILICONE & OKKLUGlas® baby AERO SILICONE provides air circulation through ventilation openings.

Warnings:

Spatial vision will be severely restricted when wearing a watch glass bandage (roadworthiness). The plaster must not be adhered to injured skin or mucous membranes.

Patient target group: Babies and children with the support of an adult. Please follow your doctor's instructions and select the variant and appropriate size.

Serious incidents occurring in connection with the product must be reported to us as the manufacturer and/or the competent authority of your member state.

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Berenbrinker Service GmbH