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OKKLUpetz®**Intended use:**

OKKLUpetz® is a ready-to-use silicone eye occlusion plaster with an all-round adhesive zone for covering the eye during occlusion therapy.

Application:

To ensure optimum adhesion, the skin should be clean, free of grease and dry.

Carefully remove the blue masking film. Place OKKLUpetz® in the middle of the eye so that the inner pad is centered over your eye and your eye is not covered by the patch. The narrow side with the notches should face the nose. Apply the patch to the skin with gentle pressure.

Please consult your doctor/orthoptist regarding the occlusion instructions, the optimum variant and the appropriate size.

Specifications for proper use:

Variation	Size	Performance features	Indications
OKKLUpetz® mini	65 x 55 mm	Covering and opaque darkening of the eye with better vision so that the eye with poorer vision is 'forced' to see and thus trained. (Occlusion therapy)	Strabismus, amblyopia, diplopia
OKKLUpetz® midi	71 x 57 mm		
OKKLUpetz® maxi	79 x 59 mm		

Contraindications:

Allergy to patches / patch intolerance

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Spatial vision is severely restricted by wearing OKKLUpetz® (ability to drive). The patch must not be applied to injured skin or mucous membranes.

The medical device is only to be worn when indicated or prescribed by an ophthalmologist/orthoptist!

 Due to the risk of increased bacterial contamination and loss of complete adhesion, this product is intended for single use only.

Patient target group:

Children with assistance from an adult during use.

Vigilance:

Serious incidents that have occurred in connection with the product must be reported to us as the manufacturer and/or the competent authority of your Member State.

Storage instructions:

 Store in a dry place  Protect from sunlight

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Berenbrinker Service GmbH